



Sentrex BioSponge MPD

Intended Use:

The Sentrex BioSponge MPD is a sterile dressing used in the management of full and partial thickness wounds, ulcers, burns, abrasions, surgical sites, and grafts. The Sentrex BioSponge MPD is indicated for wounds with moderate to high amounts of exudate and may be used in conjunction with negative pressure wound therapy (NPWT).

Instructions for Use:

1. Prepare wound bed as normal, i.e. debridement and irrigation.
2. Remove Sentrex BioSponge MPD from pouch and cut to size, if necessary.
3. Moisten with saline as desired
4. Apply Sentrex BioSponge MPD directly to the wound site
5. If used in conjunction with NPWT, cover the Sentrex BioSponge with a standard NPWT dressing/dressing kit according to manufacturer's instructions for use, paying specific attention to contraindications, precautions, and warnings. Where applicable, follow instructions for use with dressings.
6. If used without NPWT, cover with a secondary sterile bandage to secure in place.
7. Change dressing as needed. A heavily exuding wound may require frequent dressing changes.
8. The Sentrex BioSponge MPD should be changed in accordance with physician recommendations.

Precautions:

Sentrex BioSponge MPD is suitable for use under compression therapy with healthcare professional supervision.

Warnings:

Do not use if the package has been compromised. The Sentrex BioSponge MPD is for single-use only. Do not re-sterilize. Sterility is guaranteed in unopened, unadulterated packages. When used in conjunction with negative pressure wound therapy (NPWT), follow warnings for the specific negative pressure wound therapy device used.

Contraindications:

The Sentrex BioSponge MPD can not be used in patients who are allergic to chitosan. Product use should be discontinued should signs of sensitization occur. The Sentrex BioSponge MPD is not indicated for use to manage third degree burns.

Storage conditions:

Do not store the Sentrex BioSponge MPD hydrated.

Caution – U.S. federal law restricts the device to sale by or on the order of a physician.

Bionova Medical, Inc.
3011 Centre Oak Way, Suite 102
Germantown, TN 38138
info@bionovamedical.com
Ph: (901) 748-2581
www.bionovamedical.com
U.S. Pat. No. 8,993,540